



SAMPLE RETURN SHIPPING CHECKLIST

Instructions

Check each step before sealing your shipping box.

1.  Vial(s) frozen?
2.  Vial(s) inside the biohazard bag?
3.  Biohazard bag centered in blue paq ?
4.  2 frozen COLDICE* 8 oz ice packs placed inside the blue paq?
5.  Blue paq placed at the bottom of the styrofoam shipping container?
6.  2 frozen COLDICE* 8 oz ice packs placed on top of the blue paq?
7.  Styrofoam lid fully closed and taped shut?
8.  Order form and insurance paperwork filled out and included in the box?

If shipping pickup is going to be delayed, please place box in a refrigerator until shipping time.

***DO NOT USE ANY OTHER ICE PACKS OTHER THAN "COLDICE" PACKS**

 Shipped by: _____

Thank you for ensuring your sample is packed correctly for safe, compliant return shipping.