



**PLEASE FOLLOW ALL PROVIDED
INSTRUCTIONS CAREFULLY**

DO NOT throw anything away from this package including the cardboard, insulated box, or any of the other materials contained within.

Thank you!

Components of the Blue Pack System:

1. Blue Pack fabric zipper specimen insulator (Containing two internal 8 oz Cold Packs)
2. 2 x 8 oz Cold Packs
3. UN3373 Label
4. Cardboard Box containing Styrofoam insulated container



Instructions for Blue Ice Shipping System

CRITICAL: The Blue Pack system **MUST** keep the sample frozen during shipping. Please follow these instructions carefully. Not following these instructions may result in thawed samples that will not produce valid clinical results.

1 | When Shipping System is Received:

- Immediately place Blue Pack Zipper Bag and the five 3 oz. Cold Packs into the freezer. **DO NOT** remove the larger cold packs from inside of the Blue Pack.
- **CRITICAL:** These items must stay in the freezer for a minimum of 72 hours before they are used to ship a sample.
- **CRITICAL:** Do not use any of the items provided with this shipment for anything other than returning samples to EnteroTrack. Not shipping samples in the provided configuration may result in thawed samples that are no longer valid.

2 | Once clinical samples are obtained:

- Place buffer vials with clinical sample into the freezer.
- **CRITICAL:** Buffer vials need to be kept in freezer for a minimum of 24 hours to ensure sample is fully frozen.


3 | Once samples and materials are ready for shipping:

1. Remove all materials from freezer and ensure they are fully frozen.

- Buffer Vials (Containing clinical samples)
- Blue Pack Bag (with internal cold packs)
- Two 8-oz Cold Packs (supplied from EnteroTrack)


2. Place the following into the Blue Pack:

- Frozen buffer vials containing the clinical sample(s) in a biological sample bag
- Then zip up the Blue Pack

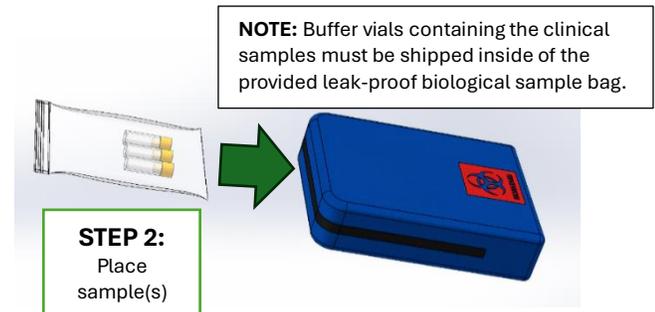

↓ THEN ↓

3. Affix the UN3373 label to the side of the shipping container.
4. Place the following items into the Insulated Container:
 - Blue Pack (containing Samples and Cold Pack)
 - Two Cold Packs
5. Place the Insulated lid onto the container.
6. Place any necessary paperwork on top of the insulated lid and tape the cardboard shut.

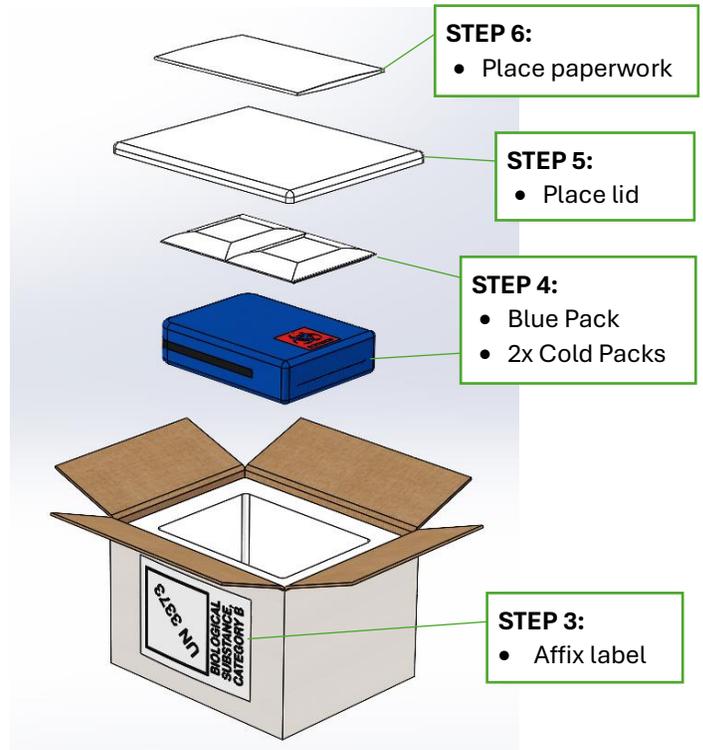

↓ THEN ↓

7. Ship **OVERNIGHT** to EnteroTrack Labs.

CRITICAL: Send an email to labs@enterotrack.com letting us know clinical samples are on their way. If shipping pickup is delayed, place the prepared box into a freezer until close to pickup time.